KU-100 NEU

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ONLY FOR DENTAL USE BY QUALIFIED PERSONNEL

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English

Indication:


Polishing paste for acrylic denture resins.

Technique:

- In order to achieve an exceptional polishing result with **KU-100 NEU** and your dentures, we recommend the following working procedure:
Grind the dentures with a sandpaper (grain size **240 µm** or **280 µm**) or rubber polishers, which have a similar grain size.
- For pre-polishing at the dental lab lathe (1st or 2nd gear) we highly recommend brushes (Ø 50 – 80 mm) with multiple rows including black bristle (medium hardness) and cotton.

To get the high-lustre surface polish with cotton maps (also 1st or 2nd gear). Here it is not necessary to apply additional paste **KU-100 NEU**.
- In case of handpiece polishing we recommend:
Rotational speed of **5.000– 20.000 rpm** and the use of goat hair brushes (Ø 15 – 21 mm, soft or medium hardness).
For high-lustre polishing use cotton buffing wheels (Ø 23 – 25 mm). Here it is not necessary to apply additional paste **KU-100 NEU**.
- Cleaning is done in a ultrasonic unit (approx. 45° - 50°C) for **1 – 2 minutes**. In the same way cleaning with a steaming device is possible.
- For the ultimate super high-lustre polishing we highly recommend **Protofix Polish Cream** (REF P8000).

Please follow the instructions of **Protofix Polish Cream**.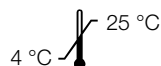

Storage:

Don't store above 25 °C. Keep away from sunlight. Keep locked up and out of reach of children. Keep container tightly closed.

Presentation:

REF

8078-200	KU-100 NEU	200 g
8078-400	KU-100 NEU	400 g