PROTOFIX POLISH CREAM

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ONLY FOR DENTAL USE BY QUALIFIED PERSONNEL

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
English

Indication:

Polish cream for super high-luster polishing of acrylic denture resins

Technique:

1. In order to achieve a super high-lustre surface on dentures after polishing with **KU 100 NEU** (REF 8078), the use of **PROTOFIX POLISH CREAM** is an absolute must for every dental technician.
2. In case of polishing at a dental lab lathe we highly recommend brushes (Ø 30 – 50 mm) with multiple rows including bristle (medium hardness) and cotton. And a low contact pressure.
To get a high-lustre surface polish with cotton maps (2nd gear). Here it is not necessary to apply additional cream **PROTOFIX POLISH CREAM**.
3. In case of handpiece polishing we recommend:
Low rotational speed and the use of goat hair brushes (Ø 15 – 21 mm, medium hardness).
For high-lustre polishing use cotton buffing wheels (Ø 23 – 25 mm). Here it is not necessary to apply additional cream **PROTOFIX POLISH CREAM**.
4. Cleaning is done in an ultrasonic unit (approx. 45° - 50°C) for **1 – 2 minutes**. In the same way cleaning with a steaming device is possible.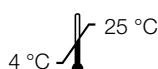

Storage:

Don't store above 25 °C. Keep away from sunlight. Keep locked up and out of reach of children. Keep container tightly closed.

Presentation:

REF

P8000 PROTOFIX POLISH CREAM 100g jar